


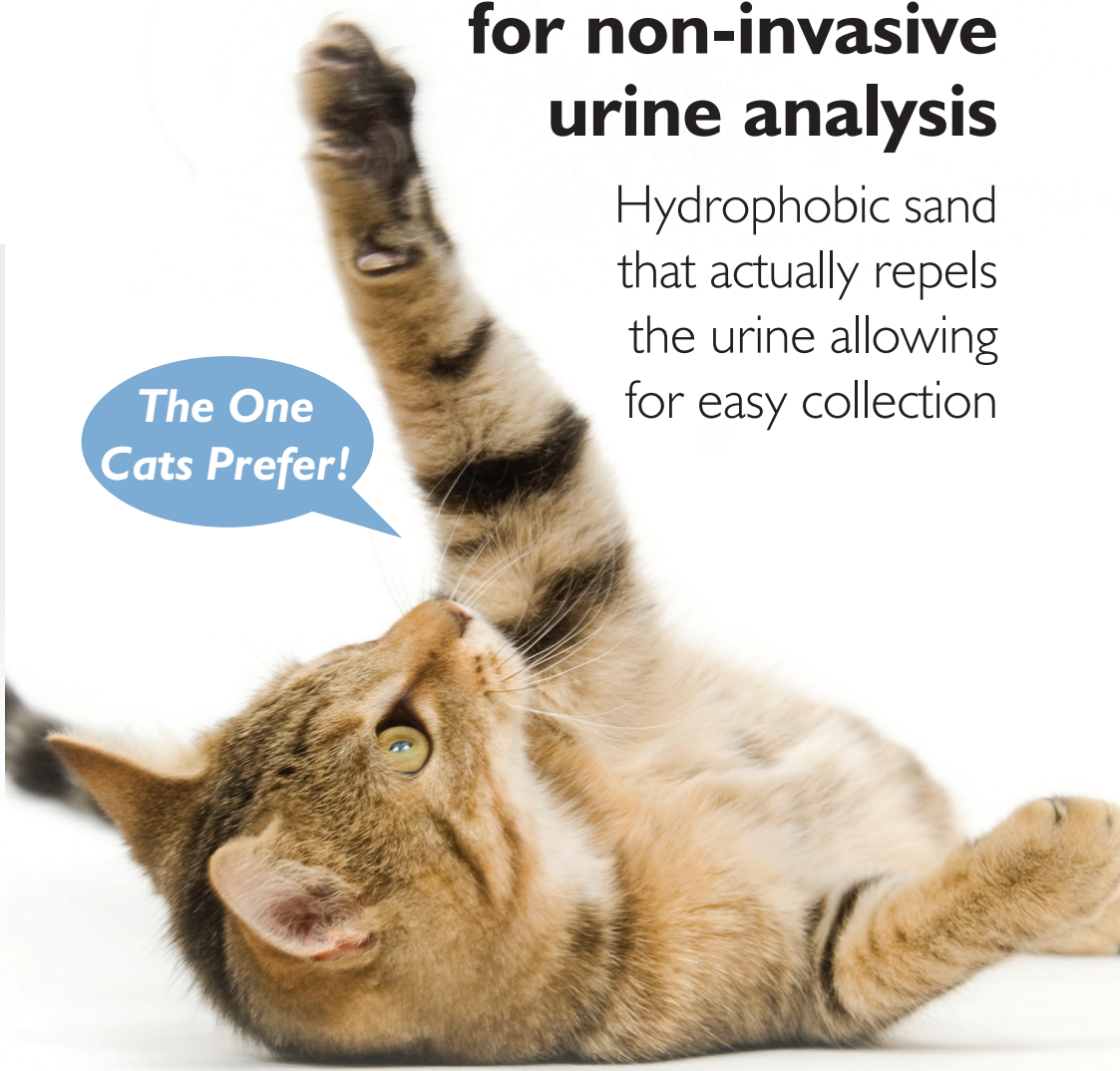

A revolutionary product for non-invasive urine analysis

Hydrophobic sand that actually repels the urine allowing for easy collection

The One Cats Prefer!



Benefits of MediCat


- Non-invasive
- Easy to use
- Cats prefer sand as litter
- Convenient kit containing all necessary utensils
- Environmentally-friendly and biodegradable



GlobalTech Intl.Ltd.

www.medicat.co.uk ☎ 020 8455 7122
email: david@MediCat.co.uk

MediCat

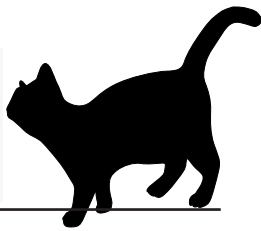

Preferred by cats

A 12-week study of hospitalised cats, comparing MediCat with a currently-used polystyrene bead product¹

Cats consistently prefer MediCat Hydrophobic Sand to the polystyrene alternative¹

Preferable by veterinary practice

Easy-to-collect samples

The sample stays on the sand surface, unlike polystyrene to which the urine tends to adhere. This is especially relevant when the urine quantity is small - commonly encountered with inflammatory urinary conditions.



Simple to use

◀ A drop of water sitting on the sand



MediCat is ideal for use by both veterinarians and cat owners

How MediCat works

- MediCat is placed in the litter tray
- The cat urinates on the hydrophobic sand
- The urine stays as a puddle on the surface
- The urine sample is collected using a pipette and transferred to a container



MediCat is supplied with a pipette and container



Typical uses for MediCat

- Monitoring crystalluria
- Monitoring urine pH
- Monitoring glycosuria
- Screening for inflammatory changes
- Monitoring urinary concentration specific gravity



Veterinarian approval

“The use of such a product will improve the ability of veterinarians to monitor the conditions of cats suffering from several chronic conditions and easily identify new patients...”¹



Ref: 1. “A new method for cats urine collection”
Clinica December 2008, no.4.